



M.U.L.E.™ PRO Cylinder Conversion Kit Instructions



The following instructions will assist you in converting your M.U.L.E. PRO to secure a MD or MJD size medical oxygen cylinder.

Cylinder Conversion Kit Components




Cylinder Divider Cylinder Pillow




M.U.L.E. PRO

1. Open the main pocket of your M.U.L.E. PRO and remove all the internal dividers and sliders. Insert the Cylinder Divider and align it with the middle bottom divider above the wheel assembly.

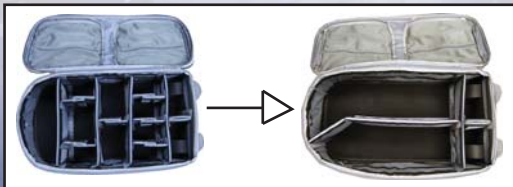

2. Insert the small slider dividers on the lower portion of the Cylinder Divider. You can customize the size of the pockets for your requirements. Insert a MD or MJD cylinder and wrap it securely with the Cylinder Pillow. 3 strips of loop (fuzzy) velcro should appear on the top of the cylinder if the pillow is installed correctly.

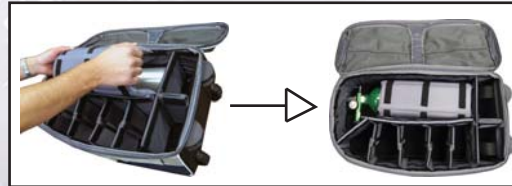

3. Congratulations, your installation is now complete. You now also have the option of adding stickit strips to the 3 strips of loop (fuzzy) velcro.