



PRO SERIES X

Cylinder Conversion Kit Instructions

The following instructions will assist you in converting your PRO series bag to secure a medical oxygen cylinder.

M8001X Cylinder Conversion Kit Components



Cylinder Pillow

Cylinder Divider

OMNI PRO X



1. Open the main pocket of your Omni and remove all of the internal dividers and sliders.



2. Insert the Cylinder Divider so that it lines up with the cylinder access panel on the left side of the main pocket.



3. Insert the small slider dividers on the lower portion of the Cylinder Divider. You can customize the size of the pockets for your team's requirements.



4. Insert medical oxygen cylinder into the upper portion of the Cylinder Divider.



5. Insert and wrap the Cylinder Pillow across the cylinder. 3 strips of loop (fuzzy) velcro should appear on the top of the cylinder if the pillow is installed correctly.



6. Congratulations, your installation is now complete. You now also have the option of adding stickit strips to the 3 strips of loop (fuzzy) velcro.

